

CryoPen® X+



Standard applicators

- Ø 1-3mm Power: █
- Ø 2-6mm Power: █ █
- Ø 4-8mm Power: █ █ █ █
- Ø 7-20mm Power: █ █ █ █ █ █

60 mm applicators

- Ø 1-3mm Power: █
- Ø 2-6mm Power: █ █



With better control of procedures the CryoPen X+ sets new standards for superb outcomes.

The CryoPen X+ dispenses a fine jet of nitrous oxide under high pressure which allows you to work with millimeter precision.

It offers the ideal solution for accurate, quick, effective stress free treatment of all skin imperfections.

Instrument

With the possibility to use both 8g and 16g cartridges and the complete range including longer applicators, CryoPen X+ gives additional value in all situations.

Potential Users

Dermatologists, GP's, Gynecologists, Urologists, Veterinarians.

Cartridges

8g N₂O cartridges: 1 box containing 24 cartridges
1 carton containing 12 boxes

16g N₂O cartridges: 1 box containing 6 cartridges
1 carton containing 12 boxes

Once punctured the pressure will remain without loss for many weeks. Shelf life 24 months. Storage at room temperature.

CryoPen X+ set with 6 applicators

Red/blue dot applicator: Ø 1-3mm
flow time 110-140sec (8g) / 250-280sec (16g)

Red/white dot applicator: Ø 2-6mm
flow time 80-100sec (8g) / 180-210sec (16g)

Red/green dot applicator: Ø 4-8mm
flow time 65-80sec (8g) / 145-165sec (16g)

Red/yellow dot applicator: Ø 7-20mm
flow time 75-100sec (16g only)

60mm red/blue dot applicator: Ø 1-3mm
flow time 110-140sec (8g) / 250-280sec (16g)

60mm red/white dot applicator: Ø 2-6mm
flow time 80-100sec (8g) / 180-210sec (16g)

What kind of lesions?

All benign lesions: Warts, actinic & seborrheic keratosis, hemangioma, molluscum contagiosum, condyloma, plantar warts, and more...

About the procedure

No special preparation is required. Treatment is applied for 2 to 25 seconds on each spot. In larger areas circular or crossing movement is required to include all tissue.