

# CryoPen® XP



|           |           |            |             |
|-----------|-----------|------------|-------------|
| ● Ø 1-3mm | ● Ø 2-4mm | ● Ø 3-5mm  | ● Ø 5-8mm   |
| Power: ■  | Power: ■■ | Power: ■■■ | Power: ■■■■ |

With better control of procedures the CryoPen XP sets new standards for superb outcomes.

The CryoPen XP dispenses a fine jet of nitrous oxide under high pressure which allows you to work with millimeter precision.

It offers the ideal solution for accurate, quick, effective, stress-free treatment of all skin imperfections.

## Instrument

With the possibility to use both 8g and 16g cartridges and the complete range including longer applicators, CryoPen XP gives additional value in all situations. Up to 15 treatments (7 minutes' flow) on one cartridge.

## Potential Users

Dermatologists, GPs, Gynecologists, Urologists, Veterinarians.

## Cartridges

8g N<sub>2</sub>O cartridges: 1 box containing 24 cartridges  
1 carton containing 12 boxes

16g N<sub>2</sub>O cartridges: 1 box containing 6 cartridges  
1 carton containing 12 boxes

Once punctured the pressure will remain without loss for many weeks. Shelf life 24 months. Storage at room temperature.

## CryoPen XP set with 4 applicators

Blue ring applicator: Ø 1-3mm  
flow time 200-215sec (8g) / 410-425sec (16g)

Grey ring applicator: Ø 2-4mm  
flow time 155-170sec (8g) / 315-330sec (16g)

Green ring applicator: Ø 3-5mm  
flow time 100-115sec (8g) / 175-190sec (16g)

Yellow ring applicator: Ø 5-8mm  
flow time 70-85 (8g) / 110-125sec (16g)

## What kind of lesions?

All benign lesions: warts, actinic & seborrheic keratosis, hemangioma, molluscum contagiosum, condyloma, plantar warts, and more...

## About the procedure

No special preparation is required. Treatment is applied for 2 to 25 seconds on each spot. In larger areas circular or crossing movement is required to include all tissue.